



1

Insert the Mouth Rest with the ridges facing left and right.



2

Turn the Mouth Rest up so the BACK teeth are positioned on the ridges.



3

Brush the teeth on the OPPOSITE side of the Mouth Rest. Reverse to brush the other side.

How to Use the Open Wide® Mouth Prop in uncooperative persons



Front teeth should NOT bite down on Mouth Rest



Note that ONLY the BACK teeth should rest on the mouth rest, NOT the front teeth.



If the person clenches, so you cannot get the Mouth Rest into the mouth, try dipping the mouth rest into applesauce or jelly before inserting.



NOTE: The Open Wide® Mouth Rest is for single patient use only; it can be cleaned with hot water or in a dishwasher. Check for integrity of the foam before after each use and replace as needed.