

# Mobile Medical Systems International Pty Ltd

Tower 2 Level 21, 201 Sussex Street, Sydney NSW 2000 AUSTRALIA  
Tel: +61 2 9252 4410 Email: [info@mobilemedical.com.au](mailto:info@mobilemedical.com.au)



Mobile Medical  
Systems

## Open Wide® Reusable Mouth Props / Instructions for Use

The SCC Open Wide® Reusable Mouth Props with Wrap Around Handle comes with a *sample disposable saliva ejector* to show proper positioning through the mouth prop 'port'.

Latex free, the head and handle are permanently bonded together for safety, contains no metal parts to interfere with radiography and can be steam sterilised. An optional 'through-hole' located within the core of the mouth prop is designed to hold a saliva ejector in place if desired.

Please dispose of the accompanying ejector after use and do not reuse or sterilise.


### Clean the Mouth Prop before Use

The Open Wide® Re-Usable Mouth Prop is not sterile when packaged and can be steam sterilised (autoclaved) at a maximum temperature of 285 F (140.5 C). *Please follow instructions provided by the autoclave manufacturer regarding recommended time required to achieve asepsis.*

### Instructions for use: Optional Use of Saliva Ejector for Dental Applications

Insert tail end of the saliva ejector (*the end that attaches to your connecting hose or suction device*) into the hole at the front of the wedge-shaped portion of the mouth prop (*tapered end*) until it exits at the rear of the prop, as shown.

Connect the end of the saliva ejector to the suction device. Adjust the tip (*mouth end*) of the saliva ejector for desired placement and discard the ejector after use; clean OWR and sterilise per protocol as noted above.

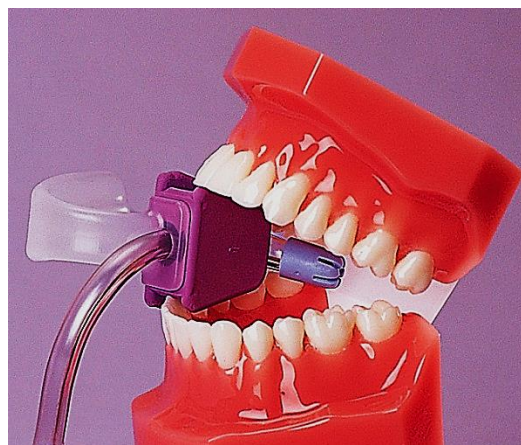

*Insert the narrow tail end of the saliva ejector into the hole at the tapered end of the prop.*