


Cleaning Procedure: Molded Foam Products

Applies to the Stay N Place® Airway Positioner (SP-AP FM)
and the Stay N Place® Lap Cushion (SP-FLEX)

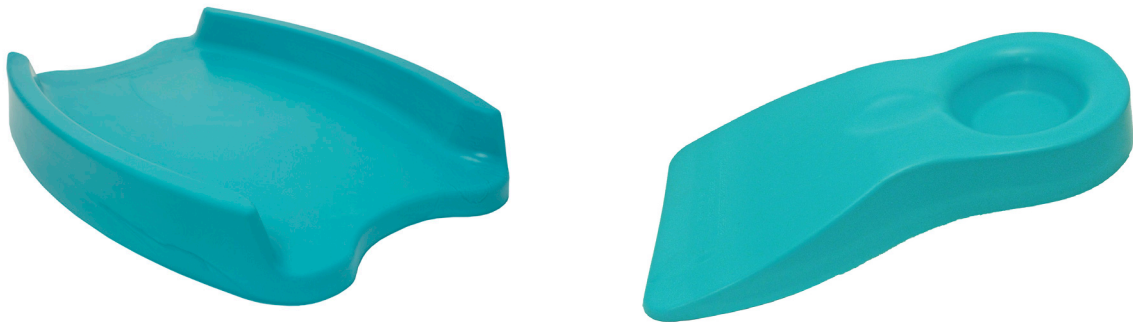

Clean surfaces of the molded foam product with a mild detergent/soap. Making sure to remove all debris, particularly biofilm material (saliva / blood, inorganic and organic matter).

Once the surface is clean, use disinfectant wipes or spray for the recommended amount of time indicated on the label of the disinfectant, then wipe dry. If a surface is not cleaned first, the success of the disinfection process can be compromised. When a surface cannot be cleaned adequately, it should be protected with barriers.

Please Note: Alcohol is a poor cleaning agent in the presence of bioburden (blood and saliva, bacterial spores.) After exposure to alcohol, denatured bioburden becomes more insoluble and tenaciously adheres to most surfaces. This may cause staining.