

Cleaning Procedure: Vinyl-Covered Rainbow® Products and Stay N Place® Products



Applies to Rainbow® Boards (RBI, RBS, RBM, RBL, RBHM, + RBHL), Rainbow® Triangle Pillow (RTP), Rainbow® Head Tilt (RHT), Rainbow® Knee Support (RKS), Rainbow® Toe Guard (RTG) and Rainbow® Head Stabilizer **cushions only* (RHII, RHIC, RHIA). As well as, our Stay N Place® Products, including Stay N Place® Pillows (SPNP, SPHC, SPKP, SPCP, + SPH), Stay N Place® Head and Neck Rest (SPHNR), Stay N Place® Booster Seat (SPBS), Stay N Place® Chair Cushion (SPCC), and the vinyl-Covered versions of our Stay N Place® Airway Positioner (SP-AP VIN) and Stay N Place® Lap Board (SP-LAP BD)

After each use, clean the entire surface with mild soap or detergent to remove any contaminants. Apply disinfectant to the surface with either a spray or wipes, following the manufacturer's directions, then wipe dry.

Do not allow disinfecting agent to remain on the vinyl, as this may degrade the material and reduce the life of the product. Avoid heat.